

Assay Performance Characteristics
ulti med Products
www.ultimed.de

June 2004

Product Name: *DrugControl*[®] Amphetamine (AMP) 1000 ng / mL

Product ID: Card 008Z111

Dipstick 008T081

Cutoff Levels:

Assay	Calibrator	Cut-off (ng/mL)
Amphetamine 1000 (AMP1000)	D-Amphetamine	1000

Specificity (sensitivity to chemically-related compounds):

	LEVEL (ng/mL)	% REACTIVITY
Amphetamine1000 (AMP1000)		
D-Amphetamine	1,000	100%
D,L-Amphetamine sulfate	3,000	33%
L-Amphetamine	50,000	2%
Phentermine	3,000	33%
D,L-3,4-Methylenedioxy-amphetamine	2,000	50%