

Assay Performance Characteristics
ulti med Products
www.ultimed.de

April 2004

Product Name: *DrugControl*[®] Cocaine (COC) 300 ng / mL

Product ID: Card 008Z114

Dipstick 008T082

Cutoff Levels:

Assay	Calibrator	Cut-off (ng/mL)
Cocaine (COC)	Benzoyllecgonine	300

Specificity (sensitivity to chemically-related compounds):

	LEVEL (ng/mL)	% REACTIVITY
COCAINE (COC)		
Benzoyllecgonine	300	100%
Cocaine HCl	780	38%
Cocaethylene	12,500	2%
Ecgonine HCl	32,000	1%