

Assay Performance Characteristics  
ulti med Products  
www.ultimed.de

April 2004

Product Name: *DrugControl*<sup>®</sup> Ecstasy / MDMA 500 ng / mL

Product ID: Card 008AC81

Dipstick 008S081

**Cutoff Levels:**

Assay	Calibrator	Cut-off (ng/mL)
(±) 3, 4-Methylenedioxymethamphetamine HCl (Ecstasy)	(±) 3, 4-Methylenedioxymethamphetamine HCl	500

**Specificity (sensitivity to chemically-related compounds):**

	LEVEL (ng/mL)	% REACTIVITY
<b>Ecstasy</b>		
(±)3,4-Methylenedioxymethamphetamine HCl (MDMA)	500	100%
(±)3,4-Methylenedioxyamphetamine HCl (MDA)	3,000	17%
3,4-Methylenedioxyethylamphetamine (MDE)	300	167%