

Assay Performance Characteristics
ulti med Products
www.ultimed.de

April 2004

Product Name: *DrugControl*[®] Metamphetamine (MET) 500 ng / mL

Product ID: Card 008A114

Dipstick n/a

Cutoff Levels:

Assay	Calibrator	Cut-off (ng/mL)
Methamphetamine (MET)	D- Methamphetamine	500

Specificity (sensitivity to chemically-related compounds):

	LEVEL (ng/mL)	% REACTIVITY
METHAMPHETAMINE (MET)		
D-Methamphetamine	500	100%
L-Methamphetamine	4,000	13%
(±)-3,4-Methylenedioxymethamphetamine	1,000	50%
Mephentermine	25,000	2%
D-Amphetamine	50,000	1%
D,L-Amphetamine	75,000	<1%
Chloroquine	12,500	4%
(1R,2S)-(-)-Ephedrine	50,000	1%
p-Hydroxymethamphetamine	15,000	3%
L-Phenylephrine	100,000	<1%
β-Phenylethylamine	75,000	<1%