

Assay Performance Characteristics
ulti med Products
www.ultimed.de

April 2004

Product Name: *DrugControl*[®] Opiates (OPI) 2000 ng / mL

Product ID: Card 008AC38

Dipstick 008AD35

Cutoff Levels:

Assay	Calibrator	Cut-off (ng/mL)
Opiates (OPI)	Morphine	2000

Specificity (sensitivity to chemically-related compounds):

	LEVEL (ng/mL)	% REACTIVITY
Opiates (OPI)		
Codeine	2,000	100%
Ethylmorphine	5,000	40%
Hydrocodone	12,500	16%
Hydromorphone	5,000	40%
Levophanol	75,000	3%
6-Monoacetylmorphine	5,000	40%
Morphine 3- β -D-glucuronide	2,000	100%
Morphine	2,000	100%
Norcodeine	12,500	16%
Normorphone	50,000	4%
Oxycodone	25,000	8%
Oxymorphone	25,000	8%
Procaine	150,000	1%
Thebaine	100,000	2%