

Assay Performance Characteristics  
ulti med Products  
www.ultimed.de

April 2004

Product Name: *DrugControl*<sup>®</sup> Tricyclic Antidepressants 1000 ng / mL

Product ID: Card 008A101

Dipstick 008A100

**Cutoff Levels:**

Assay	Calibrator	Cut-off (ng/mL)
Tricyclic Antidepressants (TCA)	Nortriptyline	1000

**Specificity (sensitivity to chemically-related compounds):**

	LEVEL (ng/mL)	% REACTIVITY
<b>Tricyclic Antidepressants (TCA)</b>		
Nortriptyline	1,000	100%
Nordoxepine	1,000	100%
Trimipramine	3,000	33%
Amitriptyline	1,500	67%
Promazine	1,500	67%
Desipramine	200	500%
Imipramine	400	250%
Clomipramine	12,500	8%
Doxepine	2,000	50%
Maprotiline	2,000	50%
Promethazine	25,000	4%