

DrugControl® Drugs of Abuse Assays Cross-Reactivity Manual by Generic Name		
COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS
Acetaminophen (See also Paracetamol)	Aceta	Non-reactive
Acetaminophen (See also Paracetamol)	Acephen	Non-reactive
Acetaminophen (See also Paracetamol)	Apacet	Non-reactive
Acetaminophen (See also Paracetamol)	Dapacen	Non-reactive
Acetaminophen (See also Paracetamol)	Feverall	Non-reactive
Acetaminophen (See also Paracetamol)	Tylenol	Non-reactive
Acetaminophen (See also Paracetamol)	Excedrin (combination)	Non-reactive
Acetaminophen (See also Paracetamol)	Panadol	Non-reactive
Acetaminophen (See also Paracetamol)	Temptra	Non-reactive
Acetaminophen with Codeine (see also Paracetamol with codeine)	Tylenol 3	Positive for Opiates (OPI, MOR)
Acetaminophen with Codeine (see also Paracetamol with codeine)	Tylenol with codeine	Positive for Opiates (OPI, MOR)
Acetophenetidin	Phenacetin	Non-reactive
Acetylsalicylic acid	Aspirin	Non-reactive
Acetylsalicylic acid	Anadin	Non-reactive
Acetylsalicylic acid	Anasin	Non-reactive
Acetylsalicylic acid	Bufferin	Non-reactive
Acetylsalicylic acid	Caprin	Non-reactive
Acetylsalicylic acid	Disprin	Non-reactive
Acetylsalicylic acid	Ecotrin	Non-reactive
Acetylsalicylic acid	Empirin	Non-reactive
Acetylsalicylic acid	Excedrin (combination)	Non-reactive
Allobarbital	No known trade names	Positive for Barbiturates (BAR)
Alphenol	No known trade names	Positive for Barbiturates (BAR)
Alprazolam	Xanax	Positive for Benzodiazepines (BZD)
Aluminum Chloride Hexahydrate	Drichlor	Non-reactive
Aluminum Chloride Hexahydrate	Anhydrol Forte	Non-reactive
Aluminum Hydroxide	Alu-Cap	Non-reactive
Aluminum Hydroxide	Alisone	Non-reactive
Aluminum Hydroxide	Gastrocote	Non-reactive
Aluminum Hydroxide	Kolanticon	Non-reactive
Aluminum Hydroxide	Maalox	Non-reactive
Aluminum Hydroxide	Maalox TC	Non-reactive
Aluminum Hydroxide	Mucogel	Non-reactive
Aluminum Hydroxide	Pyrogastrone	Non-reactive
Aluminum Hydroxide	Topal	Non-reactive
Alverine Citrate	Spasmonal	Non-reactive
Alverine Citrate	Spasmonal Fibre	Non-reactive
Aminopyrine		Non-reactive
Amitriptyline	Elavil	Positive for Tricyclic Antidepressants (TCA)
Amitriptyline	Lentizol	Positive for Tricyclic Antidepressants (TCA)
Amitriptyline	Tryptizol	Positive for Tricyclic Antidepressants (TCA)
Amitriptyline	Triptafen	Positive for Tricyclic Antidepressants (TCA)
Amitriptyline	Triptafen-M	Positive for Tricyclic Antidepressants (TCA)
Ammonia/lpecuanha	lpecac	Non-reactive
Amobarbital	Amytal	Positive for Barbiturates (BAR)
Amobarbital	Tuinal	Positive for Barbiturates (BAR)
Amoxicillin or Amoxycillin	Amoxil	Non-reactive
Amoxicillin or Amoxycillin	Amoram	Non-reactive
Amoxicillin or Amoxycillin	Augmentin	Non-reactive
Amoxicillin or Amoxycillin	Heliclear	Non-reactive
Ampicillin	Penbritin	Non-reactive
Ampicillin	Polycillin	Non-reactive
Ampicillin	Principen	Non-reactive
Antazoline Sulphate	Otrivine-Anistin	Non-reactive
Approbarital		Positive for Barbiturates (BAR)
Aspirin	Bayer Aspirin	Non-reactive
Aspirin	Excedrin	Non-reactive
Aspirin	ASA	Non-reactive
Aspirin	Angettes	Non-reactive
Aspirin	Asasantin	Non-reactive
Aspirin	Caprin	Non-reactive
Atenolol	Beta-adlat	Non-reactive
Atenolol	Co-tenidone	Non-reactive
Atenolol	Kalten	Non-reactive
Atenolol	Tenben	Non-reactive
Atenolol	Tenif	Non-reactive
Atenolol	Tenoret 50	Non-reactive
Atenolol	Tenorectic	Non-reactive
Atenolol	Tenormin	Non-reactive
Atorvastatin	Lipitor	Non-reactive
Azlocillin	Securopen	Non-reactive
Barbital		Positive for Barbiturates (BAR)
Beclometasone	AeroBec	Non-reactive
Beclometasone	AeroBec Forte	Non-reactive
Beclometasone	Asmabec	Non-reactive
Beclometasone	Beclazone	Non-reactive
Beclometasone	Becloforte	Non-reactive
Beclometasone	Becodisks	Non-reactive
Beclometasone	Beconase	Non-reactive
Beclometasone	Becotide	Non-reactive
Beclometasone	Fillair	Non-reactive
Beclometasone	Nasobec	Non-reactive
Beclometasone	Qvar	Non-reactive
Beclometasone	Ventide	Non-reactive
Beclometasone	Zonivent	Non-reactive
Bendrofluazide	Aprinox	Non-reactive
Bendrofluazide	Cogaretic	Non-reactive
Bendrofluazide	Indertic	Non-reactive
Bendrofluazide	Inderex	Non-reactive
Bendrofluazide	Neo-Naclax	Non-reactive
Bendrofluazide	Neo-Naclax-K	Non-reactive
Bendrofluazide	Prestim	Non-reactive
Bendrofluazide	Tenben	Non-reactive
Benzalkonium	Bradosol	Non-reactive
Benzylamine HCl	Diffiam	Non-reactive
Benzylpenicillin	Crystapen	Non-reactive
Bromazepam	Lexotan	Positive for Benzodiazepines (BZD)
Brompheniramine	Dimetapp	Non-reactive
Brompheniramine	Dimotapp	Non-reactive
Brompheniramine	Dimotane	Non-reactive
Buprenorphin Hydrochlorid/ Naloxone Hydrochlorid/ Bupropion	Suboxone	Positive for Buprenorphin (BUP)
Bupropion	Weilbutrin	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS
Bupropion	Zyban	Non-reactive
Butabarbital	Butisol	Positive for Barbiturates (BAR)
Butabarbital	Soneryl	Positive for Barbiturates (BAR)
Butalbital	Floricet	Positive for Barbiturates (BAR)
Butalbital	Florinal	Positive for Barbiturates (BAR)
Butethal		Positive for Barbiturates (BAR)
Canestan	Canestan ear/skin/vaginitis/HC	Non-reactive
Carbamazepine	Tegretol	Non-reactive
Carbamazepine	Teril	Non-reactive
Carbamazepine	Timonil	Non-reactive
Cephalexin	Ceporex	Non-reactive
Cephalexin	Keflex	Non-reactive
Chloral Hydrate	Welldorm	Non-reactive
Chlorazepate	Tranxene	Positive for Benzodiazepines (BZD)
Chlordiazepoxide	Librium	Positive for Benzodiazepines (BZD)
Chlorhexidine Gluconate	Bactrigras	Non-reactive
Chlorhexidine Gluconate	Cordsodyl	Non-reactive
Chlorhexidine Gluconate	Chlorohex	Non-reactive
Chlorhexidine Gluconate	CX Powder	Non-reactive
Chlorhexidine Gluconate	Dermol	Non-reactive
Chlorhexidine Gluconate	Hibicet	Non-reactive
Chlorhexidine Gluconate	Hibiscrub	Non-reactive
Chlorhexidine Gluconate	Hibisol	Non-reactive
Chlorhexidine Gluconate	Hibitane	Non-reactive
Chlorhexidine Gluconate	Instillagel	Non-reactive
Chlorhexidine Gluconate	Naseptin	Non-reactive
Chlorhexidine Gluconate	Nystaform	Non-reactive
Chlorhexidine Gluconate	Serotulle	Non-reactive
Chlorhexidine Gluconate	Steripod	Non-reactive
Chlorhexidine Gluconate	Tisept	Non-reactive
Chlorhexidine Gluconate	Unisept	Non-reactive
Chlorhexidine Gluconate	Uriflex	Non-reactive
Chlorhexidine Gluconate	Uro-Trainer	Non-reactive
Chlorpheniramine Maleate	Galpseud	Non-reactive
Chlorpheniramine Maleate	Haymine	Non-reactive
Chlorpheniramine Maleate	Pirton	Non-reactive
Chlorpromazine	Largactil	Non-reactive
Cimetidine	Dyspamet	Non-reactive
Cimetidine	Tagamet	Non-reactive
Cimetidine	Zita	Non-reactive
Clobazam	Frisium	Positive for Benzodiazepines (BZD)
Clonazepam	Clonopin	Positive for Benzodiazepines (BZD)
Clonazepam	Klonopin	Positive for Benzodiazepines (BZD)
Clonazepam	Rivotril	Positive for Benzodiazepines (BZD)
Citalopram	Cipramil	Non-reactive
Co-Amoxiclav	Augmentin	Non-reactive
Co-Amoxiclav	Augmentin Duo	Non-reactive
Codeine Phosphate	Codafen Continus	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Codeine Linctus	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Pediatric BP	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Galcodine	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Kapake	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Migraleve	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Solpadol	Positive for Opiates (OPI/MOR)
Codeine Phosphate	Tylox	Positive for Opiates (OPI/MOR)
Co-Fluampicil	Magnapen	Non-reactive
Co-Phenotrope (atropine/diphenoxylate)	Lomotil	Non-reactive
Co-Phenotrope (atropine/diphenoxylate)	Trepergen	Non-reactive
Delorazepam	Briantum	Positive for Benzodiazepines (BZD)
Dexamethasone	Decadron	Non-reactive
Dexamethasone	Dexa-Rhinaspray Duo	Non-reactive
Dexamethasone	Maxidex	Non-reactive
Dexamethasone	Maxidrol	Non-reactive
Dexamethasone	Minims	Non-reactive
Dexamethasone	Otomize	Non-reactive
Dexamethasone	Sofradex	Non-reactive
Dexamphetamine Sulphate	Adderall	Positive for Amphetamine (AMP)
Dexamphetamine Sulphate	Adderall XR	Positive for Amphetamine (AMP)
Dexamphetamine Sulphate	Dexedrine	Positive for Amphetamine (AMP)
Dextropropoxyphene	Darvon	Positive for Propoxyphene (PPX)
Dextropropoxyphene	Darvocet	Positive for Propoxyphene (PPX)
Dextropropoxyphene	Co-proximal	Positive for Propoxyphene (PPX)
Diazepam	Diazemuls	Positive for Benzodiazepines (BZD)
Diazepam	Stesolid	Positive for Benzodiazepines (BZD)
Diazepam	Valclair	Positive for Benzodiazepines (BZD)
Diazepam	Valium	Positive for Benzodiazepines (BZD)
Diclofenac Sodium	Dicloflex	Non-reactive
Diclofenac Sodium	Diclomax	Non-reactive
Diclofenac Sodium	Motifene	Non-reactive
Diclofenac Sodium	Volraman	Non-reactive
Diclofenac Sodium	Volsaid	Non-reactive
Diclofenac Sodium	Voltarol	Non-reactive
Dicyclomine	Kolanticon	Non-reactive
Dicyclomine	Merbentyl	Non-reactive
Dihydrocodeine	DHC Continus	Positive for Opiates (OPI/MOR)
Dihydrocodeine	Paramol	Positive for Opiates (OPI/MOR)
Dihydrocodeine	Remedeine	Positive for Opiates (OPI/MOR)
Dihydrocodeine	Remedeine Forte	Positive for Opiates (OPI/MOR)
Dimethicone	Asilone	Non-reactive
Dimethicone	Fancorsil	Non-reactive
Diphenhydramine	Benadryl	Non-reactive
Diphenhydramine	Medinex	Non-reactive
Diphenhydramine	Nytol	Non-reactive
Diphenhydramine	Panadol Night	Non-reactive
Diphenoxylate with Atropine	Lomotil	Non-reactive
Dothiepin	Prothiaden	Non-reactive
Doxycycline	Vibramycin	Non-reactive
Doxycycline	Vibramycin-D	Non-reactive
Doxepin	Sinequan	Positive for Tricyclic Antidepressants (TCA)
Doxepin	Xepin	Positive for Tricyclic Antidepressants (TCA)
Doxepin	Sustiva	Positive for Cannabinoids (THC)* * Urinary metabolite(s) only; parent compound is non-reactive
Efavirenz		Non-reactive
Erythromycin	Arpimycin	Non-reactive
Erythromycin	Benzmycin	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS
Erythromycin	Erycane	Non-reactive
Erythromycin	Erymax	Non-reactive
Erythromycin	Erythrocin	Non-reactive
Erythromycin	Erythroped	Non-reactive
Erythromycin	Ilosone	Non-reactive
Erythromycin	Isotrexin	Non-reactive
Erythromycin	Lactobinate	Non-reactive
Erythromycin	Stiemycin	Non-reactive
Erythromycin	Tiloryth	Non-reactive
Erythromycin	Zineryl	Non-reactive
Estazolam	ProSom	Positive for Benzodiazepines (BZD)
Ethambutol	Myambutol	Non-reactive
Ethylmorphine		Positive for Opiates (OPI/MOR)
Flucloxacillin	Co-fluampicil	Non-reactive
Flucloxacillin	Floxapen	Non-reactive
Flucloxacillin	Magnapen	Non-reactive
Flunitrazepam	Rohypnol	Positive for Benzodiazepines (BZD)
Fluoxetine	Prozac	Non-reactive
Fluoxetine	Sarafem	Non-reactive
Flupentixol Decanoate	Depixol	Non-reactive
Flupentixol Decanoate	Fluanxol	Non-reactive
Fusidic Acid (Sodium Fusidate)	Fucithalmic	Non-reactive
Fusidic Acid (Sodium Fusidate)	Fucibet	Non-reactive
Fusidic Acid (Sodium Fusidate)	Fucidin	Non-reactive
Gentamicin Sulphate	Cidomycin	Non-reactive
Gentamicin Sulphate	Geniticin	Non-reactive
Gentamicin Sulphate	Genisone	Non-reactive
Hydrocodone	Lorcet	Positive for Opiates (OPI/MOR)
Hydrocodone	Loratab	Positive for Opiates (OPI/MOR)
Hydrocodone	Vicodin	Positive for Opiates (OPI/MOR)
Hydrocortisone	Actinac	Non-reactive
Hydrocortisone	Alphaderm	Non-reactive
Hydrocortisone	Alphosyl-HC	Non-reactive
Hydrocortisone	Anugesic-HC	Non-reactive
Hydrocortisone	Anusol-HC	Non-reactive
Hydrocortisone	Calmurid-HC	Non-reactive
Hydrocortisone	Canesten-HC	Non-reactive
Hydrocortisone	Colioam	Non-reactive
Hydrocortisone	Daktacort	Non-reactive
Hydrocortisone	Dioderm	Non-reactive
Hydrocortisone	Econacort	Non-reactive
Hydrocortisone	Efortelan	Non-reactive
Hydrocortisone	Efcortesel	Non-reactive
Hydrocortisone	Eurax-HC	Non-reactive
Hydrocortisone	Gregoderm	Non-reactive
Hydrocortisone	Hydrocortistab	Non-reactive
Hydrocortisone	Hydrocortone	Non-reactive
Hydrocortisone	Mildison lipocream	Non-reactive
Hydrocortisone	Neo-cortef	Non-reactive
Hydrocortisone	Nystaform	Non-reactive
Hydrocortisone	Perinal	Non-reactive
Hydrocortisone	Proctofoam	Non-reactive
Hydrocortisone	Quinocort	Non-reactive
Hydrocortisone	Terra-cortil	Non-reactive
Hydrocortisone	Timodine	Non-reactive
Hydrocortisone	Uniroid-HC	Non-reactive
Hydrocortisone	Vioform-HC	Non-reactive
Hydrocortisone	Xyloproct	Non-reactive
Hydrocortisone-17-butyrate	Locoid	Non-reactive
Hydrocortisone-17-butyrate	Locoid C	Non-reactive
Hydromorphone	Dilaudid	Positive for Opiates (OPI/MOR)
Hydromorphone	Hydrostat	Positive for Opiates (OPI/MOR)
Hydroxychloroquine	Plaquenil	Non-reactive
Hydroxyzine Hydrochloride	Atarax	Non-reactive
Hydroxyzine Hydrochloride	Ucerax	Non-reactive
Hyoscine butylbromide	Buscopan	Non-reactive
Hyoscine butylbromide	Scopoderm	Non-reactive
Ibuprofen	Brufen	Non-reactive
Ibuprofen	Codafen	Non-reactive
Ibuprofen	Fenbid	Non-reactive
Ibuprofen	Ibugel	Non-reactive
Ibuprofen	Ibuspray	Non-reactive
Ibuprofen	Motrin	Non-reactive
Ibuprofen	Proflex	Non-reactive
Indomethacin	Flexin continus	Non-reactive
Indomethacin	Indocid	Non-reactive
Indomethacin	Indomod	Non-reactive
Interferon Alfa	Roferon-A	Non-reactive
Interferon Alfa	Intron A	Non-reactive
Interferon Alfa	Viraferon	Non-reactive
Interferon Alfa	Wellferon	Non-reactive
Isoniazide	INH	Non-reactive
Isosorbide Dinitrate	Angitak	Non-reactive
Isosorbide Dinitrate	Cedocard	Non-reactive
Isosorbide Dinitrate	Isocard	Non-reactive
Isosorbide Dinitrate	Isoket	Non-reactive
Isosorbide Dinitrate	Isordil	Non-reactive
Isosorbide Dinitrate	Sorbichew	Non-reactive
Isosorbide Dinitrate	Sorbitrate	Non-reactive
Isosorbide Dinitrate	Sorbid SA	Non-reactive
Koalin and Morphine Mixture	Diocalm	Positive for Opiates (OPI/MOR)
Koalin and Morphine Mixture	Entersan	Positive for Opiates (OPI/MOR)
Koalin and Morphine Mixture	Opazimes	Positive for Opiates (OPI/MOR)
Ketoprofen	Orudis	Non-reactive
Ketoprofen	Oruvail	Non-reactive
Ketoprofen	Powergel	Non-reactive
Lactulose	Lactugal	Non-reactive
Lactulose	Duphalec	Non-reactive
Lamotrigine	Lamital	Non-reactive
Lansoprazole	Heliclear	Non-reactive
Lansoprazole	Zoton	Non-reactive
Lisinopril HCl	Carace	Non-reactive
Lisinopril HCl	Zestril	Non-reactive
Lisinopril HCl	Zestoretic	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS
Lofepramine	Gamanil	Positive for Tricyclic Antidepressants (TCA)
Lofepramine	Lomont	Positive for Tricyclic Antidepressants (TCA)
Lofexidine	Britoflex	Non-reactive
Loperamide HCl	Imodium	Non-reactive
Loperamide HCl	Loperagen	Non-reactive
Loperamide HCl	Norimide	Non-reactive
Loratadine	Claritin	Non-reactive
Lorazepam	Ativan	Positive for Benzodiazepines (BZD)
Lormetazepam	Noctamide	Positive for Benzodiazepines (BZD)
Magnesium Alginate	Gaviscon	Non-reactive
Mannitol	Gaviscon	Non-reactive
Medazepam	Anxitol	Positive for Benzodiazepines (BZD)
Medazepam	Lerisum	Positive for Benzodiazepines (BZD)
Medazepam	Medacepan	Positive for Benzodiazepines (BZD)
Medazepam	Nobritol	Positive for Benzodiazepines (BZD)
Medazepam	Nobrium	Positive for Benzodiazepines (BZD)
Mefenamic Acid	Ponstan	Non-reactive
Mefenamic Acid	Ponstel	Non-reactive
Meperidine	Demerol	Non-reactive
Meperidine	Pethidine	Non-reactive
Metoclopramide	Reglan	Non-reactive
Methadone Hydrochloride	Dolophine	Positive for Methadone (MTD)
Methadone Hydrochloride	Methadose	Positive for Methadone (MTD)
Methadone Hydrochloride	Physetone	Positive for Methadone (MTD)
d-Methamphetamine HCL	Desoxyn	Positive for Methamphetamine (MET)
d-Methamphetamine HCL	Methedrine	Positive for Methamphetamine (MET)
d-Methamphetamine HCL	Methamprex	Positive for Methamphetamine (MET)
l-Methamphetamine HCL		
Methylenedioxyamphetamine (MDA)	Eve (slang)	Positive for Amphetamine (AMP) and Ecstasy (MDMA)
Methylenedioxyamphetamine (MDA)	Love Drug (slang)	Positive for Amphetamine (AMP) and Ecstasy (MDMA)
Methylenedioxyamphetamine (MDMA)	Ecstasy (slang)	Positive for Methamphetamine (MET) and Ecstasy (MDMA)
Methylenedioxymethamphetamine (MDMA)	XTC (slang)	Positive for Methamphetamine (MET) and Ecstasy (MDMA)
Methylenedioxymethamphetamine (MDMA)	Adam (slang)	Positive for Methamphetamine (MET) and Ecstasy (MDMA)
Methylenedioxymethamphetamine (MDMA)	E (slang)	Positive for Methamphetamine (MET) and Ecstasy (MDMA)
Metronidazole	Anabact	Non-reactive
Metronidazole	Elyzol	Non-reactive
Metronidazole	Flagyl	Non-reactive
Metronidazole	Metrogel	Non-reactive
Metronidazole	Metrolyl	Non-reactive
Metronidazole	Metrotop	Non-reactive
Metronidazole	Neuratop	Non-reactive
Metronidazole	Noritate	Non-reactive
Metronidazole	Rozex	Non-reactive
Metronidazole	Zidoval	Non-reactive
Metronidazole	Zymoet	Non-reactive
Morphine	Astramorph	Positive for Opiates (MOR, OPI)
Morphine	Cyclimorph	Positive for Opiates (MOR, OPI)
Morphine	Duramorph	Positive for Opiates (MOR, OPI)
Morphine	Morcap	Positive for Opiates (MOR, OPI)
Morphine	Morphine Sulfate	Positive for Opiates (MOR, OPI)
Morphine	MS Contin	Positive for Opiates (MOR, OPI)
Morphine	Oramorph	Positive for Opiates (MOR, OPI)
Morphine	Roxanol	Positive for Opiates (MOR, OPI)
Morphine	Severedol	Positive for Opiates (MOR, OPI)
Nadolol	Corgard	Non-reactive
Nadolol	Corqaretic	Non-reactive
Nadolol	Corqartic	Non-reactive
Naloxone	Narcan	Non-reactive
Naloxone Hydrochlorid/ Buprenorphin Hydrochlorid	Suboxon	Positive for Buprenorphin (BUP)
Naltrexone	Antaxone	Non-reactive
Naltrexone	Nalorex	Non-reactive
Naltrexone	Trexan	Non-reactive
Naproxen	Aleve	Non-reactive
Naproxen	Condrotec	Non-reactive
Naproxen	Naprtec	Non-reactive
Naproxen	Naprosen	Non-reactive
Naproxen	Naprosyn	Non-reactive
Naproxen	Nycopren	Non-reactive
Naproxen	Synflex	Non-reactive
Nicotine	Nicoderm	Non-reactive
Nicotine	Nicorette	Non-reactive
Nicotine	Nicotinell	Non-reactive
Nicotine	Nicotrol	Non-reactive
Nicotine	Niquitin	Non-reactive
Nifedipine	Adalat	Non-reactive
Nifedipine	Nifecard	Non-reactive
Nifedipine	Nifar	Non-reactive
Nifedipine	Nif-ten	Non-reactive
Nifedipine	Procardia	Non-reactive
Nitrazepam	Mogadon	Positive for Benzodiazepines (BZD)
Nitrazepam	Somnite	Positive for Benzodiazepines (BZD)
Norethindrone	BiNovum	Non-reactive
Norethindrone	Brevinor	Non-reactive
Norethindrone	Climagest	Non-reactive
Norethindrone	Climesse	Non-reactive
Norethindrone	Elleste Duet	Non-reactive
Norethindrone	Estracombi	Non-reactive
Norethindrone	Evorel	Non-reactive
Norethindrone	Kliofem	Non-reactive
Norethindrone	Kliovance	Non-reactive
Norethindrone	Loestrin	Non-reactive
Norethindrone	Micronor	Non-reactive
Norethindrone	Norlutin	Non-reactive
Norethindrone	Noriday	Non-reactive
Norethindrone	Norimin	Non-reactive
Norethindrone	Norinyl	Non-reactive
Norethindrone	Noristat	Non-reactive
Norethindrone	Nouvelle	Non-reactive
Norethindrone	Ovysmen	Non-reactive
Norethindrone	TriNovum	Non-reactive
Norethindrone	Trisquens	Non-reactive
Norethindrone	Utovian	Non-reactive
Noscapine	Narcotine	Non-reactive
d/l-Octopamine	No known trade names	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS
Olanzapine	Zyprexa	Non-reactive
Orphenadrine	Norfex	Non-reactive
Orphenadrine	Norgesic	Non-reactive
Orphenadrine	Disipil	Non-reactive
Oxazepam	Serax	Positive for Benzodiazepines (BZD)
Oxazepam	Ox-pam	Positive for Benzodiazepines (BZD)
Oxymetazoline	Afrin	
Oxymetazoline	Neo-synephrine	
Oxytetracycline	Terra-Cortil	Non-reactive
Oxytetracycline	Terramycin	Non-reactive
Oxytetracycline	Trimovate	Non-reactive
Paracetamol (Acetaminophen)	Fortagesic	Non-reactive
Paracetamol (Acetaminophen)	Paradote	Non-reactive
Paracetamol (Acetaminophen) /Codeine Preparations	Tylenol 3	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Co-codamol	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Codafen	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Co-dydramol	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Kapake	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Remedine	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Solpadol	Positive for Opiates (MOR, OPI)
Paracetamol (Acetaminophen) /Codeine Preparations	Tylex	Positive for Opiates (MOR, OPI)
Paroxetine	Paxil	Non-reactive
Paroxetine	Seroxat	Non-reactive
Penicillin	Combicillin	Non-reactive
Penicillin	Mefoxin	Non-reactive
Pentobarbital	Nembutal	Positive for Barbiturates (BAR)
Pericyazine	Amplan	Non-reactive
Pericyazine	Aolept	Non-reactive
Pericyazine	Apamin	Non-reactive
Pericyazine	Iryakin	Non-reactive
Pericyazine	Nemactil	Non-reactive
Pericyazine	Neulactil	Non-reactive
Pericyazine	Propetyl	Non-reactive
Pericyazine	Psycholept	Non-reactive
Phenobarbitone (see also Phenobarbital)	Luminal	Positive for Barbiturates (BAR)
Phenytoin	Dilantin	Non-reactive
Phenytoin	Epanutin	Non-reactive
Phenytoin	Epitard	Non-reactive
Phenobarbital	Donnatal	Positive for Barbiturates (BAR)
Pholcodine	Galenphol	Positive for Opiates (MOR, OPI)
Pholcodine	Strong BP	Positive for Opiates (MOR, OPI)
Pholcodine	Pavacol-D	Positive for Opiates (MOR, OPI)
Pholcodine	Thebacon	Positive for Opiates (MOR, OPI)
Piperacillin	Pipracil	Non-reactive
Piperacillin	Tazobactam	Non-reactive
Prazepam	Centrax	Positive for Benzodiazepines (BZD)
Prazepam	Demetrin	Positive for Benzodiazepines (BZD)
Prednisolone	Delta-cortef	Non-reactive
Prednisolone	Econopred	Non-reactive
Prednisolone	Inflamase	Non-reactive
Prednisolone	Pediapred	Non-reactive
Prednisolone	Prelone	Non-reactive
Procaine	Novocain	Positive for Opiates (OPI, MOP)
Progesterol		Non-reactive
Promethazine	Allerfen	Non-reactive
Promethazine	Anergan	Non-reactive
Promethazine	Antinaus	Non-reactive
Promethazine	Aprobit	Non-reactive
Promethazine	Avomine	Non-reactive
Promethazine	Baymethzine	Non-reactive
Promethazine	Diprozin	Non-reactive
Promethazine	Fenergan	Non-reactive
Promethazine	Methazine	Non-reactive
Promethazine	Phenergan	Non-reactive
Propranolol	Beta-Progane	Non-reactive
Propranolol	Inderal	Non-reactive
Propranolol	Inderetic	Non-reactive
Propranolol	Inderex	Non-reactive
Pseudoephedrine	Afrinol	Non-reactive
Pseudoephedrine	Sudafed	Non-reactive
Pseudoephedrine	Tylenol Cold (combination)	Non-reactive
Ranitidine	Pylorid, Zantac	Positive for Methamphetamine (MET)* *Urinary metabolite(s) only; parent compound is non-reactive
Salbutamol	Aerocrom	Non-reactive
Salbutamol	Aerolin	Non-reactive
Salbutamol	Airomir	Non-reactive
Salbutamol	Asmasal	Non-reactive
Salbutamol	Combivent	Non-reactive
Salbutamol	Duovent	Non-reactive
Salbutamol	Ventide	Non-reactive
Salbutamol	Ventodisks	Non-reactive
Salbutamol	Venotlin	Non-reactive
Salbutamol	Volmax	Non-reactive
Secobarbital	Seconal	Positive for Barbiturates (BAR)
Sennosides	Senokot	Non-reactive
Sertraline	Zolof	Non-reactive
Sodium Valproate	Depakene	Non-reactive
Sodium Valproate	Depakote	Non-reactive
Sodium Valproate	Epilim	Non-reactive
Temazepam	Restoril	Positive for Benzodiazepines (BZD)
Testosterone	Andropatch	Non-reactive
Testosterone	Restandol	Non-reactive
Testosterone	Sustanon	Non-reactive
Testosterone	Viomone	Non-reactive
Testosterone	slang name "anabolic steroids"	Non-reactive
Thioridazine Hydrochloride	Mellari	Non-reactive
Thyroxine Sodium	Eltroxin	Non-reactive
Tramadol	Ultram	Non-reactive
Tramadol	Tramake	Non-reactive
Tramadol	Zamadol	Non-reactive
Tramadol	Zydol	Non-reactive
Trazodone	Desyrel	Non-reactive
Trazodone	Molipaxin	Non-reactive
Trazodone	Trialodine	Non-reactive

COMPOUND (Generic Name)	COMPOUND (Trade Name)	RESULTS
Triazolam	Halcion	Positive for Benzodiazepines (BZD)
Venlafaxine	Effexor	Potential Positive for PCP
Venlafaxine	Effexor XL	Potential Positive for PCP
Warfarin Sodium	Coumadin	Non-reactive
Warfarin Sodium	Marevan	Non-reactive

ulti med Products (Deutschland) GmbH
Reeshoop 1 22926 Ahrensburg
Telefon: 04102 - 80090
Fax: 04102 - 50082
e-mail: info@ultimed.de

ulti med Products (Belgium) BVBA
Honzebroekstraat 137 8800 Roeselare
Phone: +32 +51 200 425
Fax: +32 +51 200 449
e-mail: belgium@ultimed.org

Februar 2008 / B