

DrugControl® Drugs of Abuse Assays Cross-Reactivity Manual by Trade Name

COMPOUND (Trade Name)	COMPOUND (Generic Name)	RESULTS
Acephen	Acetaminophen (See also Paracetamol)	Non-reactive
Aceta	Acetaminophen (See also Paracetamol)	Non-reactive
Actinac	Hydrocortisone	Non-reactive
Adalat	Nifedipine	Non-reactive
Adam (slang)	Methylenedioxymethamphetamine (MDMA)	Positive for Methamphetamine (MET) and Ecstasy (MDMA)
Adderall	Dexamphetamine Sulphate	Positive for Amphetamine (AMP)
Adderall XR	Dexamphetamine Sulphate	Positive for Amphetamine (AMP)
AeroBec	Beclometasone	Non-reactive
AeroBec Forte	Beclometasone	Non-reactive
Aerocrom	Salbutamol	Non-reactive
Aerolin	Salbutamol	Non-reactive
Afrin	Oxymetazoline	Non-reactive
Afrinol	Pseudoephedrine	Non-reactive
Airomir	Salbutamol	Non-reactive
Aleve	Naproxen	Non-reactive
Alisone	Aluminum Hydroxide	Non-reactive
Allerfen	Promethazine	Positive for Tricyclic Antidepressants (TCA)
Alphaderm	Hydrocortisone	Non-reactive
Alphosyl-HC	Hydrocortisone	Non-reactive
Alu-Cap	Aluminum Hydroxide	Non-reactive
Amoram	Amoxicillin or Amoxycillin	Non-reactive
Amoxil	Amoxicillin or Amoxycillin	Non-reactive
Amplan	Pericyazine	Non-reactive
Amytal	Amobarbital	Positive for Barbiturates (BAR)
Anabact	Metronidazole	Non-reactive
Anadin	Acetylsalicylic acid	Non-reactive
Anasin	Acetylsalicylic acid	Non-reactive
Andropatch	Testosterone	Non-reactive
Anergan	Promethazine	Positive for Tricyclic Antidepressants (TCA)
Angettes	Aspirin	Non-reactive
Angitak	Isosorbide Dinitrate	Non-reactive
Anhydrol Forte	Aluminum Chloride Hexahydrate	Non-reactive
Antaxone	Naltrexone	Non-reactive
Antinaus	Promethazine	Positive for Tricyclic Antidepressants (TCA)
Anugesc-HC	Hydrocortisone	Non-reactive
Anusol-HC	Hydrocortisone	Non-reactive
Anxitol	Medazepam	Positive for Benzodiazepines (BZD)
Aolept	Pericyazine	Non-reactive
Apacet	Acetaminophen (See also Paracetamol)	Non-reactive
Apamin	Pericyazine	Non-reactive
Aprinox	Bendrofluazide	Non-reactive
Aprobit	Promethazine	Positive for Tricyclic Antidepressants (TCA)
Arpimycin	Erythromycin	Non-reactive
ASA	Aspirin	Non-reactive
Asasantin	Aspirin	Non-reactive
Asilone	Dimethicone	Non-reactive
Asmabec	Beclometasone	Non-reactive
Asmasal	Salbutamol	Non-reactive
Aspirin	Acetylsalicylic acid	Non-reactive
Astramorph	Morphine	Positive for Opiates (MOR, OPI)
Atarax	Hydroxyzine Hydrochloride	Non-reactive
Ativan	Lorazepam	Positive for Benzodiazepines (BZD)
Augmentin	Amoxicillin or Amoxycillin	Non-reactive
Augmentin	Co-Amoxiclav	Non-reactive
Augmentin Duo	Co-Amoxiclav	Non-reactive
Avomine	Promethazine	Positive for Tricyclic Antidepressants (TCA)
Bactrigras	Chlorhexidine Gluconate	Non-reactive
Bayer Aspirin	Aspirin	Non-reactive
Baymethzine	Promethazine	Positive for Tricyclic Antidepressants (TCA)
Beclazone	Beclometasone	Non-reactive
Becloforte	Beclometasone	Non-reactive
Becodisks	Beclometasone	Non-reactive
Beconase	Beclometasone	Non-reactive
Becotide	Beclometasone	Non-reactive
Benadryl	Diphenhydramine	Non-reactive
Benzymycin	Erythromycin	Non-reactive
Beta-adlat	Atenolol	Non-reactive
Beta-Progane	Propranolol	Non-reactive
BiNovum	Norethindrone	Non-reactive
Bradosol	Benzalkonium	Non-reactive
Brevinor	Norethindrone	Non-reactive
Briantum	Delorazepam	Positive for Benzodiazepines (BZD)
Britoflex	Lofexidine	Non-reactive
Brufen	Ibuprofen	Non-reactive
Bufferin	Acetylsalicylic acid	Non-reactive
Busocopan	Hyoscine butylbromide	Non-reactive
Butisol	Butabarbital	Positive for Barbiturates (BAR)
Calmurid-HC	Hydrocortisone	Non-reactive
Canestan ear/skin/vaginitis/HC	Canestan	Non-reactive
Canesten-HC	Hydrocortisone	Non-reactive
Caprin	Acetylsalicylic acid	Non-reactive
Caprin	Aspirin	Non-reactive
Carace	Lisinopril HCl	Non-reactive
Cedocard	Isosorbide Dinitrate	Non-reactive
Centrax	Prazepam	Positive for Benzodiazepines (BZD)
Ceporex	Cephalexin	Non-reactive
Chlorohex	Chlorhexidine Gluconate	Non-reactive
Cidomycin	Gentamicin Sulphate	Non-reactive
Cipramil	Citalopram	Non-reactive
Claritin	Loratadine	Non-reactive
Climagest	Norethindrone	Non-reactive
Climesse	Norethindrone	Non-reactive
Clonopin	Clonazepam	Positive for Benzodiazepines (BZD)
Co-codamol	Paracetamol (Acetaminophen) /Codeine Preparations	Positive for Opiates (MOR, OPI)
Codafen	Ibuprofen	Non-reactive
Codafen	Paracetamol (Acetaminophen) /Codeine Preparations	Positive for Opiates (MOR, OPI)
Codafen Continus	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Codeine Linctus	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Co-dydramol	Paracetamol (Acetaminophen) /Codeine Preparations	Positive for Opiates (MOR, OPI)
Co-fluapicil	Flucloxacillin	Non-reactive

COMPOUND (Trade Name)	COMPOUND (Generic Name)	RESULTS
Cogaretic	Bendrofluazide	Non-reactive
Colioam	Hydrocortisone	Non-reactive
Combicillin	Penicillin	Non-reactive
Combivent	Salbutamol	Non-reactive
Condrotec	Naproxen	Non-reactive
Co-proximal	Dextropropoxyphene	Positive for Propoxyphene (PPX)
Cordsodyl	Chlorhexidine Gluconate	Non-reactive
Corgard	Nadolol	Non-reactive
Corgaretic	Nadolol	Non-reactive
Corgartic	Nadolol	Non-reactive
Co-tenidone	Atenolol	Non-reactive
Coumadin	Warfarin Sodium	Non-reactive
Crystapen	Benzylpenicillin	Non-reactive
CX Powder	Chlorhexidine Gluconate	Non-reactive
Cyclimorph	Morphine	Positive for Opiates (MOR, OPI)
Daktacort	Hydrocortisone	Non-reactive
Dapacen	Acetaminophen (See also Paracetamol)	Non-reactive
Darvocet	Dextropropoxyphene	Positive for Propoxyphene (PPX)
Darvon	Dextropropoxyphene	Positive for Propoxyphene (PPX)
Decadron	Dexamethasone	Non-reactive
Delta-cortef	Prednisolone	Non-reactive
Demerol	Meperidine	Non-reactive
Demetrin	Przepam	Positive for Benzodiazepines (BZD)
Depakene	Sodium Valproate	Non-reactive
Depakote	Sodium Valproate	Non-reactive
Depixol	Flupentixol Decanoate	Non-reactive
Dermol	Chlorhexidine Gluconate	Non-reactive
Desoxyin	d-Methamphetamine HCL	Positive for Methamphetamine (MET)
Desyrel	Trazodone	Non-reactive
Dexa-Rhinaspray Duo	Dexamethasone	Non-reactive
Dexedrine	Dexamphetamine Sulphate	Positive for Amphetamine (AMP)
DHC Continus	Dihydrocodeine	Positive for Opiates (OPI/MOR)
Diazemuls	Diazepam	Positive for Benzodiazepines (BZD)
Dicloflex	Diclofenac Sodium	Non-reactive
Diclomax	Diclofenac Sodium	Non-reactive
Diffiam	Benzylamine HCl	Non-reactive
Dilantin	Phenytoin	Non-reactive
Dilaudid	Hydromorphone	Positive for Opiates (OPI/MOR)
Dimetapp	Brompheniramine	Non-reactive
Dimotane	Brompheniramine	Non-reactive
Dimotapp	Brompheniramine	Non-reactive
Diocalm	Koalin and Morphine Mixture	Positive for Opiates (OPI/MOR)
Dioderm	Hydrocortisone	Non-reactive
Diprozin	Promethazine	Non-reactive
Disipil	Orphenadrine	Non-reactive
Disprin	Acetylsalicylic acid	Non-reactive
Dolophine	Methadone Hydrochloride	Positive for Methadone (MTD)
Donnatal	Phenobarbital	Positive for Barbiturates (BAR)
Drichlor	Aluminum Chloride Hexahydrate	Non-reactive
Duovent	Salbutamol	Non-reactive
Duphalec	Lactulose	Non-reactive
Duramorph	Morphine	Positive for Opiates (MOR, OPI)
Dyspamet	Cimetidine	Non-reactive
E (slang)	Methylenedioxyamphetamine (MDMA)	Positive for Methamphetamine (MET) and Ecstasy (MDMA)
Econacort	Hydrocortisone	Non-reactive
Econopred	Prednisolone	Non-reactive
Ecotrin	Acetylsalicylic acid	Non-reactive
Ecstasy (slang)	Methylenedioxyamphetamine (MDMA)	Positive for Methamphetamine (MET) and Ecstasy (MDMA)
Efortesol	Hydrocortisone	Non-reactive
Effexor	Venlafaxine	Potential Positive for PCP
Effexor XL	Venlafaxine	Potential Positive for PCP
Efortelan	Hydrocortisone	Non-reactive
Elavil	Amitriptyline	Positive for Tricyclic Antidepressants (TCA)
Elleste Duet	Norethindrone	Non-reactive
Eltroxin	Thyroxine Sodium	Non-reactive
Elyzol	Metronidazole	Non-reactive
Empirin	Acetylsalicylic acid	Non-reactive
Entersan	Koalin and Morphine Mixture	Positive for Opiates (OPI/MOR)
Epanutin	Phenytoin	Non-reactive
Epilim	Sodium Valproate	Non-reactive
Epitard	Phenytoin	Non-reactive
Erycane	Erythromycin	Non-reactive
Erymax	Erythromycin	Non-reactive
Erythrocin	Erythromycin	Non-reactive
Erythroped	Erythromycin	Non-reactive
Estracombi	Norethindrone	Non-reactive
Eurax-HC	Hydrocortisone	Non-reactive
Eve (slang)	Methylenedioxyamphetamine (MDA)	Positive for Amphetamine (AMP) and Ecstasy (MDMA)
Evorel	Norethindrone	Non-reactive
Excedrin	Aspirin	Non-reactive
Excedrin (combination)	Acetaminophen (See also Paracetamol)	Non-reactive
Excedrin (combination)	Acetylsalicylic acid	Non-reactive
Fancorsil	Dimethicone	Non-reactive
Fenbid	Ibuprofen	Non-reactive
Fenergan	Promethazine	Positive for Tricyclic Antidepressants (TCA)
Feverall	Acetaminophen (See also Paracetamol)	Non-reactive
Fillair	Beclometasone	Non-reactive
Fioricet	Butalbital	Positive for Barbiturates (BAR)
Fiorinal	Butalbital	Positive for Barbiturates (BAR)
Flagyl	Metronidazole	Non-reactive
Flexin continus	Indomethacin	Non-reactive
Floxapen	Flucloxacillin	Non-reactive
Fluanxol	Flupentixol Decanoate	Non-reactive
Fortagesic	Paracetamol (Acetaminophen)	Non-reactive
Frisium	Clobazam	Positive for Benzodiazepines (BZD)
Fucibet	Fusidic Acid (Sodium Fusidate)	Non-reactive
Fucidin	Fusidic Acid (Sodium Fusidate)	Non-reactive
Fucithalamic	Fusidic Acid (Sodium Fusidate)	Non-reactive
Galcodine	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Galenphol	Pholcodine	Positive for Opiates (MOR, OPI)
Galpseud	Chlorpheniramine Maleate	Non-reactive
Gamanil	Lofepamine	Positive for Tricyclic Antidepressants (TCA)

COMPOUND (Trade Name)	COMPOUND (Generic Name)	RESULTS
Gastrocote	Aluminum Hydroxide	Non-reactive
Gaviscon	Magnesium Alginate	Non-reactive
Gaviscon	Mannitol	Non-reactive
Genticin	Gentamicin Sulphate	Non-reactive
Gentisone	Gentamicin Sulphate	Non-reactive
Gregoderm	Hydrocortisone	Non-reactive
Halcion	Triazolam	Positive for Benzodiazepines (BZD)
Haymine	Chlorpheniramine Maleate	Non-reactive
Heliclear	Amoxicillin or Amoxycillin	Non-reactive
Heliclear	Lansoprazole	Non-reactive
Hibicet	Chlorhexidine Gluconate	Non-reactive
Hibiscrub	Chlorhexidine Gluconate	Non-reactive
Hibisol	Chlorhexidine Gluconate	Non-reactive
Hibitane	Chlorhexidine Gluconate	Non-reactive
Hydrocortistab	Hydrocortisone	Non-reactive
Hydrocortone	Hydrocortisone	Non-reactive
Hydrostat	Hydromorphone	Positive for Opiates (OPI/MOR)
Ibugel	Ibuprofen	Non-reactive
Ibuspray	Ibuprofen	Non-reactive
Ilosone	Erythromycin	Non-reactive
Imodium	Loperamide HCl	Non-reactive
Inderal	Propranolol	Non-reactive
Inderetic	Propranolol	Non-reactive
Inderex	Bendrofluazide	Non-reactive
Inderex	Propranolol	Non-reactive
Indertic	Bendrofluazide	Non-reactive
Indocid	Indomethacin	Non-reactive
Indomod	Indomethacin	Non-reactive
Inflamase	Prednisolone	Non-reactive
INH	Isoniazide	Non-reactive
Instillagel	Chlorhexidine Gluconate	Non-reactive
Intron A	Interferon Alfa	Non-reactive
Ipecac	Ammonia/Ipecacuanha	Non-reactive
Iryakin	Pericyazine	Non-reactive
Isocard	Isosorbide Dinitrate	Non-reactive
Isoket	Isosorbide Dinitrate	Non-reactive
Isordil	Isosorbide Dinitrate	Non-reactive
Isotrexin	Erythromycin	Non-reactive
Kalten	Atenolol	Non-reactive
Kapake	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Kapake	Paracetamol (Acetaminophen) /Codeine Preparations	Positive for Opiates (MOR, OPI)
Keflex	Cephalexin	Non-reactive
Kliefem	Norethindrone	Non-reactive
Klovance	Norethindrone	Non-reactive
Klonopin	Clonazepam	Positive for Benzodiazepines (BZD)
Kolanticon	Aluminum Hydroxide	Non-reactive
Kolanticon	Dicyclomine	Non-reactive
Lactobinate	Erythromycin	Non-reactive
Lactugal	Lactulose	Non-reactive
Lamital	Lamotrigine	Non-reactive
Largactil	Chlorpromazine	Non-reactive
Lentizol	Amitriptyline	Positive for Tricyclic Antidepressants (TCA)
Lerisum	Medazepam	Positive for Benzodiazepines (BZD)
Lexotan	Bromazepam	Positive for Benzodiazepines (BZD)
Librium	Chlordiazepoxide	Positive for Benzodiazepines (BZD)
Lipitor	Atorvastatin	Non-reactive
Locoid	Hydrocortisone-17-butyrate	Non-reactive
Locoid C	Hydrocortisone-17-butyrate	Non-reactive
Loestrin	Norethindrone	Non-reactive
Lomont	Lofepamine	Positive for Tricyclic Antidepressants (TCA)
Lomotil	Co-Phenotrope (atropine/diphenoxylate)	Non-reactive
Lomotil	Diphenoxylate with Atropine	Non-reactive
Loperagen	Loperamide HCl	Non-reactive
Loratab	Hydrocodone	Positive for Opiates (OPI/MOR)
Lorcet	Hydrocodone	Positive for Opiates (OPI/MOR)
Love Drug (slang)	Methylenedioxymphetamine (MDA)	Positive for Amphetamine (AMP) and Ecstasy (MDMA)
Luminal	Phenobarbitone (see also Phenobarbital)	Positive for Barbiturates (BAR)
Maalox	Aluminum Hydroxide	Non-reactive
Maalox TC	Aluminum Hydroxide	Non-reactive
Magnapen	Flucloxacillin	Non-reactive
Magnapen	Co-Fluampicil	Non-reactive
Marevan	Warfarin Sodium	Non-reactive
Maxidex	Dexamethasone	Non-reactive
Maxidrol	Dexamethasone	Non-reactive
Medacepan	Medazepam	Positive for Benzodiazepines (BZD)
Medinex	Diphenhydramine	Non-reactive
Mefoxin	Penicillin	Non-reactive
Mellaril	Thioridazine Hydrochloride	Non-reactive
Merbentyl	Dicyclomine	Non-reactive
Methadose	Methadone Hydrochloride	Positive for Methadone (MTD)
Methamprex	d-Methamphetamine HCL	Positive for Methamphetamine (MET)
Methazine	Promethazine	Positive for Tricyclic Antidepressants (TCA)
Methedrine	d-Methamphetamine HCL	Positive for Methamphetamine (MET)
Metrogel	Metronidazole	Non-reactive
Metrolyl	Metronidazole	Non-reactive
Metrotop	Metronidazole	Non-reactive
Micronor	Norethindrone	Non-reactive
Migravele	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Milidison lipocream	Hydrocortisone	Non-reactive
Minims	Dexamethasone	Non-reactive
Mogadon	Nitrazepam	Positive for Benzodiazepines (BZD)
Molipaxin	Trazodone	Non-reactive
Morcap	Morphine	Positive for Opiates (MOR, OPI)
Morphine Sulfate	Morphine	Positive for Opiates (MOR, OPI)
Motifene	Diclofenac Sodium	Non-reactive
Motrin	Ibuprofen	Non-reactive
MS Contin	Morphine	Positive for Opiates (MOR, OPI)
Mucogel	Aluminum Hydroxide	Non-reactive
Myambutol	Ethambutol	Non-reactive
Nalorex	Naltrexone	Non-reactive
Naprtec	Naproxen	Non-reactive
Naprosen	Naproxen	Non-reactive

COMPOUND (Trade Name)	COMPOUND (Generic Name)	RESULTS
Naprosyn	Naproxen	Non-reactive
Narcan	Naloxone	Non-reactive
Narcotine	Noscapine	Non-reactive
Naseptin	Chlorhexidine Gluconate	Non-reactive
Nasobec	Beclometasone	Non-reactive
Nemactil	Pericyazine	Non-reactive
Nembutal	Pentobarbital	Positive for Barbiturates (BAR)
Neo-cortef	Hydrocortisone	Non-reactive
Neo-Naclax	Bendrofluazide	Non-reactive
Neo-Naclax-K	Bendrofluazide	Non-reactive
Neo-synephrine	Oxymetazoline	
Neulactil	Pericyazine	Non-reactive
Neuratop	Metronidazole	Non-reactive
Nicoderm	Nicotine	Non-reactive
Nicorette	Nicotine	Non-reactive
Nicotinell	Nicotine	Non-reactive
Nicotrol	Nicotine	Non-reactive
Nifar	Nifedipine	Non-reactive
Nifecard	Nifedipine	Non-reactive
Nif-ten	Nifedipine	Non-reactive
Niquitin	Nicotine	Non-reactive
No known trade names	Allobarbitol	Positive for Barbiturates (BAR)
No known trade names	Alphenol	Positive for Barbiturates (BAR)
No known trade names	d/l-Octopamine	Non-reactive
Nobritol	Medazepam	Positive for Benzodiazepines (BZD)
Nobrium	Medazepam	Positive for Benzodiazepines (BZD)
Noctamide	Lormetazepam	Positive for Benzodiazepines (BZD)
Norflex	Orphenadrine	Non-reactive
Norgesic	Orphenadrine	Non-reactive
Noriday	Norethindrone	Non-reactive
Norimide	Loperamide HCl	Non-reactive
Norimin	Norethindrone	Non-reactive
Norinyl	Norethindrone	Non-reactive
Noristat	Norethindrone	Non-reactive
Noritrate	Metronidazole	Non-reactive
Nortlutin	Norethindrone	Non-reactive
Novocain	Procaine	Positive for Opiates (OPI, MOP)
Nuvelle	Norethindrone	Non-reactive
Nycopren	Naproxen	Non-reactive
Nystaform	Chlorhexidine Gluconate	Non-reactive
Nystaform	Hydrocortisone	Non-reactive
Nytol	Diphenhydramine	Non-reactive
Opazimes	Koalin and Morphine Mixture	Positive for Opiates (OPI/MOR)
Oramorph	Morphine	Positive for Opiates (MOR, OPI)
Orudis	Ketoprofen	Non-reactive
Oruvail	Ketoprofen	Non-reactive
Otomize	Dexamethasone	Non-reactive
Otrivine-Anistin	Antazoline Sulphate	Non-reactive
Ovysmen	Norethindrone	Non-reactive
Ox-pam	Oxazepam	Positive for Benzodiazepines (BZD)
Panadol	Acetaminophen (See also Paracetamol)	Non-reactive
Panadol Night	Diphenhydramine	Non-reactive
Paradote	Paracetamol (Acetaminophen)	Non-reactive
Paramol	Dihydrocodeine	Positive for Opiates (OPI/MOR)
Pavacol-D	Pholcodine	Positive for Opiates (MOR, OPI)
Paxil	Paroxetine	Non-reactive
Pediapred	Prednisolone	Non-reactive
Pediatric BP	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Penbritin	Ampicillin	Non-reactive
Perinal	Hydrocortisone	Non-reactive
Pethidine	Meperidine	Non-reactive
Phenacetin	Acetophenetidin	Non-reactive
Phenergen	Promethazine	Non-reactive
Physetone	Methadone Hydrochloride	Positive for Methadone (MTD)
Pipracil	Piperacillin	Non-reactive
Piriton	Chlorpheniramine Maleate	Non-reactive
Plaquenil	Hydroxychloroquine	Non-reactive
Polycillin	Ampicillin	Non-reactive
Ponstan	Mefenamic Acid	Non-reactive
Ponstel	Mefenamic Acid	Non-reactive
Powergel	Ketoprofen	Non-reactive
Prelone	Prednisolone	Non-reactive
Prestim	Bendrofluazide	Non-reactive
Principen	Ampicillin	Non-reactive
Procardia	Nifedipine	Non-reactive
Proctofoam	Hydrocortisone	Non-reactive
Proflex	Ibuprofen	Non-reactive
Propetyl	Pericyazine	Non-reactive
ProSom	Estazolam	Positive for Benzodiazepines (BZD)
Prothiaden	Dothiepin	Non-reactive
Prozac	Fluoxetine	Non-reactive
Psycholept	Pericyazine	Non-reactive
Pylorid	Ranitidine	Positive for Methamphetamine (MET)* *Urinary metabolite(s) only; parent compound is non-reactive
Pyrogastrone	Aluminum Hydroxide	Non-reactive
Quinocort	Hydrocortisone	Non-reactive
Qvar	Beclometasone	Non-reactive
Reglan	Metoclopramide	Non-reactive
Remedeine	Dihydrocodeine	Positive for Opiates (OPI/MOR)
Remedeine Forte	Dihydrocodeine	Positive for Opiates (OPI/MOR)
Remedine	Paracetamol (Acetaminophen) /Codeine Preparations	Positive for Opiates (MOR, OPI)
Restandol	Testosterone	Non-reactive
Restoril	Temazepam	Positive for Benzodiazepines (BZD)
Rivotril	Clonazepam	Positive for Benzodiazepines (BZD)
Roferon-A	Interferon Alfa	Non-reactive
Rohypnol	Flunitrazepam	Positive for Benzodiazepines (BZD)
Roxanol	Morphine	Positive for Opiates (MOR, OPI)
Rozex	Metronidazole	Non-reactive
Sarafem	Fluoxetine	Non-reactive
Scopoderm	Hyoscine butylbromide	Non-reactive
Seconal	Secobarbital	Positive for Barbiturates (BAR)
Securopen	Azlocillin	Non-reactive

COMPOUND (Trade Name)	COMPOUND (Generic Name)	RESULTS
Senokot	Sennosides	Non-reactive
Serax	Oxazepam	Positive for Benzodiazepines (BZD)
Serotulle	Chlorhexidine Gluconate	Non-reactive
Seroxat	Paroxetine	Non-reactive
Severedol	Morphine	Positive for Opiates (MOR, OPI)
Sinequan	Doxepin	Positive for Tricyclic Antidepressants (TCA)
slang name "anabolic steroids"	Testosterone	Non-reactive
Sofradex	Dexamethasone	Non-reactive
Solpadol	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Solpadol	Paracetamol (Acetaminophen) /Codeine Preparations	Positive for Opiates (MOR, OPI)
Somnite	Nitrazepam	Positive for Benzodiazepines (BZD)
Soneryl	Butabarbital	Positive for Barbiturates (BAR)
Sorbichew	Isosorbide Dinitrate	Non-reactive
Sorbid SA	Isosorbide Dinitrate	Non-reactive
Sorbitrate	Isosorbide Dinitrate	Non-reactive
Spasmonal	Alverine Citrate	Non-reactive
Spasmonal Fibre	Alverine Citrate	Non-reactive
Steripod	Chlorhexidine Gluconate	Non-reactive
Stesolid	Diazepam	Positive for Benzodiazepines (BZD)
Stiemycin	Erythromycin	Non-reactive
Strong BP	Pholcodine	Positive for Opiates (MOR, OPI)
Suboxone	Naloxone Hydrochlorid/ Buprenorphin Hydrochlorid	Positive for Buprenorphine (BUP)
Sudafed	Pseudoephedrine	Non-reactive
Sustanon	Testosterone	Non-reactive
Sustiva	Efavirenz	Positive for Cannabinoids (THC)* * Urinary metabolite(s) only; parent compound is non-reactive
Synflex	Naproxen	Non-reactive
Tagamet	Cimetidine	Non-reactive
Tazobactam	Piperacillin	Non-reactive
Tegretol	Carbamazepine	Non-reactive
Temgesic	Buprenorphine	Non-reactive
Tempra	Acetaminophen (See also Paracetamol)	Non-reactive
Tenben	Atenolol	Non-reactive
Tenben	Bendrofluazide	Non-reactive
Tenif	Atenolol	Non-reactive
Tenoretic	Atenolol	Non-reactive
Tenoret 50	Atenolol	Non-reactive
Tenormin	Atenolol	Non-reactive
Teril	Carbamazepine	Non-reactive
Terra-cortil	Hydrocortisone	Non-reactive
Terra-Cortil	Oxytetracycline	Non-reactive
Terramycin	Oxytetracycline	Non-reactive
Thebacon	Pholcodine	Positive for Opiates (MOR, OPI)
Tiloryth	Erythromycin	Non-reactive
Timodine	Hydrocortisone	Non-reactive
Timonil	Carbamazepine	Non-reactive
Tisept	Chlorhexidine Gluconate	Non-reactive
Topal	Aluminum Hydroxide	Non-reactive
Tramake	Tramadol	Non-reactive
Tranxene	Chlorazepate	Positive for Benzodiazepines (BZD)
Trepergen	Co-Phenotrope (atropine/diphenoxylate)	Non-reactive
Trexan	Naltrexone	Non-reactive
Trialodine	Trazodone	Non-reactive
Trimovate	Oxytetracycline	Non-reactive
TriNovum	Norethindrone	Non-reactive
Triptafen	Amitriptyline	Positive for Tricyclic Antidepressants (TCA)
Triptafen-M	Amitriptyline	Positive for Tricyclic Antidepressants (TCA)
Trisquens	Norethindrone	Non-reactive
Tryptizol	Amitriptyline	Positive for Tricyclic Antidepressants (TCA)
Tuinal	Amobarbital	Positive for Barbiturates (BAR)
Tylenol	Acetaminophen (See also Paracetamol)	Non-reactive
Tylenol 3	Acetaminophen with Codeine (see also Paracetamol with codeine)	Positive for Opiates (OPI, MOR)
Tylenol 3	Paracetamol (Acetaminophen) /Codeine Preparations	Positive for Opiates (MOR, OPI)
Tylenol Cold (combination)	Pseudoephedrine	Non-reactive
Tylenol with codeine	Acetaminophen with Codeine (see also Paracetamol with codeine)	Positive for Opiates (OPI, MOR)
Tylox	Codeine Phosphate	Positive for Opiates (OPI/MOR)
Tylox	Paracetamol (Acetaminophen) /Codeine Preparations	Positive for Opiates (MOR, OPI)
Ucerax	Hydroxyzine Hydrochloride	Non-reactive
Ultram	Tramadol	Non-reactive
Uniroid-HC	Hydrocortisone	Non-reactive
Unisept	Chlorhexidine Gluconate	Non-reactive
Uriflex	Chlorhexidine Gluconate	Non-reactive
Uro-Trainer	Chlorhexidine Gluconate	Non-reactive
Utovian	Norethindrone	Non-reactive
Valclair	Diazepam	Positive for Benzodiazepines (BZD)
Valium	Diazepam	Positive for Benzodiazepines (BZD)
Venotlin	Salbutamol	Non-reactive
Ventide	Beclometasone	Non-reactive
Ventide	Salbutamol	Non-reactive
Ventodisks	Salbutamol	Non-reactive
Vibramycin	Doxycycline	Non-reactive
Vibramycin-D	Doxycycline	Non-reactive
Vicodin	Hydrocodone	Positive for Opiates (OPI/MOR)
Vioform-HC	Hydrocortisone	Non-reactive
Viraferon	Interferon Alfa	Non-reactive
Viormone	Testosterone	Non-reactive
Volmax	Salbutamol	Non-reactive
Volraman	Diclofenac Sodium	Non-reactive
Volsaid	Diclofenac Sodium	Non-reactive
Voltarol	Diclofenac Sodium	Non-reactive
Wellbutrin	Bupropion	Non-reactive
Welldorm	Chloral Hydrate	Non-reactive
Wellferon	Interferon Alfa	Non-reactive
Xanax	Alprazolam	Positive for Benzodiazepines (BZD)
Xepin	Doxepin	Positive for Tricyclic Antidepressants (TCA)
XTC (slang)	Methylenedioxymethamphetamine (MDMA)	Positive for Methamphetamine (MET) and Ecstasy (MDMA)
Xyloproct	Hydrocortisone	Non-reactive
Zamadol	Tramadol	Non-reactive
Zantac	Ranitidine	Positive for Methamphetamine (MET)* *Urinary metabolite(s) only; parent compound is non-reactive
Zestoretic	Lisinopril HCl	Non-reactive
Zestril	Lisinopril HCl	Non-reactive
Zidoval	Metronidazole	Non-reactive

COMPOUND (Trade Name)	COMPOUND (Generic Name)	RESULTS
Zineryl	Erythromycin	Non-reactive
Zita	Cimetidine	Non-reactive
Zoloft	Sertraline	Non-reactive
Zonivent	Beclometasone	Non-reactive
Zoton	Lansoprazole	Non-reactive
Zyban	Bupropion	Non-reactive
Zydol	Tramadol	Non-reactive
Zymoet	Metronidazole	Non-reactive
Zyprexa	Olanzapine	Non-reactive
	Aminopyrine	Non-reactive
	Aprobarbital	Positive for Barbiturates (BAR)
	Barbital	Positive for Barbiturates (BAR)
	Butethal	Positive for Barbiturates (BAR)
	Ethylmorphine	Positive for Opiates (OPI/MOR)
Vick's Inhaler	I-Methamphetamine HCL	Positive for Methamphetamine (MET)
	Progesterol	Non-reactive

ulti med Products (Deutschland) GmbH
Reeshoop 1 22926 Ahrensburg
Telefon: 04102 - 80090
Fax: 04102 - 50082
e-mail: info@ultimed.de

ulti med Products (Belgium) BVBA
Honzebroekstraat 137 8800 Roeselare
Phone: +32 +51 200 425
Fax: +32 +51 200 449
e-mail: belgium@ultimed.org

Februar 2008 /B